

The HemoFlow™ Blood Collection System

ASSISTING YOU WITH BLOOD DONATIONS BY AUTOMATICALLY MIXING THE BAG (MODEL 300), MONITORING FOR SLOW FLOW RATES, CLAMPING OFF FLOW AT THE END OF THE COLLECTION AND ALLOWING YOU TO EASILY PROGRAM THE COLLECTION VOLUME.

easy to set up and use

- Easy, one minute set up.
- Simple, two-step calibration.
- Auto-tare feature simplifies the use of different bag sets.
- New users can learn how to use the HemoFlow quickly, with just three basic steps:
 - 1) Place the bag, 2) Insert the tubing, and 3) Press the START key.
- Interactive CD-ROM training and assessment program provides step by step lessons for new users.

outstanding battery life: 4 weeks* / 5 months*

The HemoFlow's long battery life keeps maintenance low while providing a cost-effective power source for your facility's group of mixers. The HemoFlow 300 Rechargeable Model requires charging only once a month.

* The HemoFlow Model 200 battery life is between 3-4 months. All battery life estimates are based on average use. Complete technical specifications are listed at www.HemoFlow.com.

specifically designed for mobile use

- Lightweight, portable and durable.
- Off the floor bed mount and floor stand mounting options.
- Precise leveling is not required.
- Transport cases are available.

affordable upgrade solutions

The HemoFlow Model 200 Monitor provides an affordable solution to upgrading from traditional scales and trip mechanisms.

Like the Model 300, the Model 200 includes features that help prevent overbleeds, such as automatic clamping at the end of the draw, low flow monitoring and programmable collection volume.



one of the lightest mixers on the market

Weighing about 4 lbs, the HemoFlow Model 300 Monitor/Mixer simplifies transport and handling. It is a perfect choice for blood banks that require automatic mixing and mobility.

Unique rocking action provides thorough mixing to increase successful units.

The Model 300 integrates the latest technology with innovative design, making it one of the most portable mixers on the market. We've combined reliability and affordability to provide you with the best possible solution to your blood banking needs.



increased savings

Number of Collections Using 1 HemoFlow	80 per week, 3120 per year
Value of a Processed Bag of Blood	\$150
Typical Loss %	5%
Loss % Using the HemoFlow	2%
Savings	3%
Bags Saved	94 (3120 X 3%)
Savings Per Year*	\$14,100

* Savings are based on the HemoFlow's ability to decrease the number of shorts and overs. This example is shown for illustration only, and will vary for individual facilities based on a variety of factors.

By using the HemoFlow Blood Collection System, more bags can be completed successfully with less phlebotomist attendance. The HemoFlow improves efficiency and provides a cost-effective solution; increasing total output by decreasing losses.

To learn more about setting up your free evaluation, call us at 1-866-HEMOFLO or visit www.HemoFlow.com.