

HEMOFLOW™ MODEL 200 QUICK REFERENCE CARD



▶ DUAL KEY FUNCTIONS

Short Press: A momentary press of the key.

Long Press: Press for 3 seconds, until the display changes.

▶ DAILY SETUP

1. Check draw volume: Short press **VOL** key.
2. Perform calibration.

▶ DIRECTIONS FOR CALIBRATION

1. Remove bag from hook.
2. Long press **SCALE** until **CAL -0-** is displayed.
3. When **CAL 500** is displayed, hang the 500 gram weight.
4. **SCL** is displayed when the calibration is complete.

▶ DIRECTIONS FOR USE

1. Hang the bag on the bag hook (25 gram min).
2. Press clamp release button & insert tubing in clamp.
3. Check that tubing loops freely from bag.
4. Short press **START**. Do not touch bag for 5 seconds.
5. Perform venepuncture (*after pressing START*).
6. Open clamp to allow flow to begin.
7. When flow starts, monitoring will begin and volume will be displayed.
8. Manually mix bag periodically per protocol.
9. Tubing is automatically clamped at end of draw. **End** is displayed.
10. Manually clamp or seal tubing per protocol.
11. Press clamp release button and remove tubing and bag.

WARNING: Do not disturb bag during last 20mls of draw.

* For further detail, refer to the HemoFlow™ Model 200 Operator's Guide.

▶ ALARMS

Lo Flo	Low flow rate. Correct low flow.
bAG Err	No bag or full bag at start. Use new bag.
Job Err (at start)	Incorrect startup. Press SCALE and complete draw by weight. Will not auto-clamp.
Job Err (at end)	Overbleed. Evaluate if bag is usable.
14'59 Err	Timeout error. Evaluate if bag is usable.
End	Draw is complete
Lo bAT	Replace with 4 "AA" batteries at end of day.

▶ KEY FUNCTIONS

START

1. Start draw: Short press when new bag is on hook.
2. Show draw time: Short press during draw. (min-sec).
3. Change time limit: Long press with no bag on hook.

ALARM

(Power Off)

1. Mute alarm sound: Short press.
2. Power off (no bag on hook): Short press.

SCALE

(Cal)

This function is disabled during draw.

1. Scale mode: Short press shows weight (0-1000gm).
2. Calibration mode: Long press with no weight on hook.

FLOW

(0-9)

Show flow rate in draw.

0-2 slow
3-7 normal
8-9 fast

VOL

1. Check draw volume (not in draw): Short press.
2. Change draw volume (not in draw): Long press.
3. Show actual draw volume (during draw): Short press.